

Disposable masks do NOT replace: frequent hand washing and social distancing (6ft)



Supply and how to wear the disposable mask

(* You will receive 1 disposable mask per shift, or additional if needed (loss, wet, dirty, etc.)

Wash your hands with soap and water for 20 seconds

Disposable mask should fully cover mouth and nose

Secure with ear loops



How to remove the disposable mask

Remove the disposable mask from behind (do not touch the front of the mask)

Wrap the disposable mask in a clean paper towel, if it is going to be reused

Discard the gloves

Wash your hands with soap and water for 20 seconds

(* If available.

Employees can choose to wear a personal cloth mask on a voluntary basis.



How to care for the disposable mask

Avoid touching the disposable mask while wearing it; if you do, wash your hands for 20 seconds

When not in use or prior to eating, please wrap the disposable mask in a clean paper towel

A disposable paper towel may be used inside of the disposable mask to create extra filter layer to extend the disposable mask's usage life



How to discard the disposable mask

Disposable masks should be discarded at the end of the shift or before, if wet, dirty or if it is damaged

Discard the disposable mask in a trash can

Discard the gloves

Wash your hands with soap and water for 20 seconds

